

VeriDoc | VeriDoc 2C

LABELLING AND DOCUMENTATION SYSTEM



VeriDoc and **VeriDoc 2C*** are the new systems for safe and professional labelling and documentation of sterile barrier systems in hospitals as well as doctors' and dentists' surgeries.

The centre piece of these systems is a screenless Print-server made by hawo (PrintBox). Using this compact PrintBox, the data read by the **ValiScan 2D** barcode scanner is transmitted to the **ValiPrint** label printer and printed. No additional computer screen or keyboard is required to record important packaging and approval information and print it onto a label. The labels can be used for sealable pouches and reels, wrapped sets or reusable containers. The **VeriDoc** systems are thus the perfect supplements to existing sealing devices.

*** NEW for VeriDoc 2C:**

A second ValiPrint label printer can be optionally connected to the new VeriDoc 2C. This means that 2 printers with 2 different labels (e.g. STEAM and VH202) can be operated with one system. The VeriDoc 2C systems also have a USB port, via which the process data for electronic documentation and tracking purposes or for statistical analysis can be transferred to a USB stick or external hard drive.

VeriDoc | VeriDoc 2C

INTEGRATING ALL STERILE BARRIER SYSTEMS

Regardless of whether it's sealable pouches and reels, wrapped sets or trays or reusable containers, **VeriDoc** and **VeriDoc 2C** assist medical professionals to meet packaging labelling requirements. It also includes sterilization inspection using a process indicator along with formal storage approval after a visual inspection, as well as documentation in the patient's file.

After the required visual inspection of the packaging, the system is informed whether or not it has passed by scanning the corresponding Data Matrix code ('sterile barrier system approved' or 'sterile barrier system not approved'). Depending on the result of the visual inspection, a label is printed with the corresponding note. The packaging may not be used if an error (e.g. open or broken seal or closure system) is detected. All required information such as batch number, name of the packer, product information or the expiry date is simply transferred to the labels via the easy-to-use **ValiScan 2D** barcode scanner and an individualized Data Matrix code list. After the sterilization, the integrated process indicator (ISO 11140-1, Class 1) shows whether or not the packaged instrument has been sterilized. After sterilization, personalized approval can be given for storage using the field provided. After the instrument has been stored and used, the so-called duplex-label can be removed from the sterile barrier system and placed in an appendix to the patient's file. This enables the user to immediately determine that each instrument, set or container being used has been properly packaged, sterilized and approved for storage. Instead of the approval field, all data can also be displayed as QR code. The data on the label can be easily transferred to electronic documentation systems or patient's file in this way.



Label with QR code for transferral of the data to documentation systems.



01

Apply the label to the sterile barrier system:

- > sealable pouches or reels
- > wrapped sets
- > reusable containers

02

After use, the duplex label can be removed from the sterile barrier system ...

03

... and applied in the appendix to the patient's file.

ValiPrint LABEL PRINTERS

Together with the practical indicator labels, it also provides helpful instrument labelling that complies with legal requirements.

The following labels are available:

- Steam (STEAM)
- Steam / Ethylene oxide (STEAM/ETO) combination label
- Steam / Formaldehyde (STEAM/FORM) combination label
- Plasma (VH2O2)

VeriDoc 2C can be equipped with a second ValiPrint label printer. This means that 2 different labels (e.g. STEAM and VH202) can be operated with one system. The printer housing has an antibacterial coating and the control panel contains a tested antibacterial substance.



hawo GmbH
Obere Au 2-4
74847 Obrigheim/Germany
T +49 (0) 6261 / 9770-0
F +49 (0) 6261 / 9770-69
VeriDoc@hawo.com
www.hawo.com